## Preservative Free Saline

### Table of Contents

EN.....	2
BG.....	4
CS.....	6
DA.....	8
DE.....	10
ES.....	12
ET.....	16
FI.....	18
FR.....	20
EL.....	<b>Error! Bookmark not defined.</b>
HU.....	22
IT.....	24
NL.....	26
NO.....	28
PL.....	30
PT.....	32
RU.....	34
SK.....	36
SL.....	38
SV.....	40
TR.....	42

CooperVision  
Hamble, SO31 4RF, UK  
www.coopervision.com

CooperVision CL Kft.  
Gorcsev Iván street 7. Building C.  
2360 Gyál, Hungary

## EN

Instructions for Use

### IMPORTANT

Read Instructions carefully before use

PRESERVATIVE FREE

SALINE SOLUTION

SUITABLE FOR RINSING ALL CONTACT LENSES

**IMPORTANT:** Do not change from the solutions recommended by your contact lens practitioner. Preservative Free Saline is not for use as a disinfectant.

Indications for use:

Preservative Free Saline is an isotonic, sterile solution, buffered to pH7.

USES:

- For the rehydration of all soft contact lenses prior to disinfection.
- As a final rinse for all contact lenses before re-insertion into the eyes.
- For dissolving tablets where saline buffered to a pH of 7 is recommended.
- For rinsing contact lens storage cases.

Intended users: For contact lens wearers.

Contra-indications: Do not use if allergic to any ingredient.

The sterile barrier for solution is the unopened bottle cap.

Disposal: use standard waste, recycle whenever possible.

### **DIRECTIONS FOR USE:**

Wash, rinse and dry your hands thoroughly. Unscrew cap and pour required amount of Preservative Free Saline solution. Solution is ready to use.

### **DISSOLVING TABLETS**

Preservative Free Saline may be used to dissolve protein remover tablets where saline buffered to a pH of 7 is recommended.

Follow the directions for use on the tablet pack.

PRECAUTIONS:

**DO NOT MIX WITH OTHER FLUIDS EXCEPT AS DIRECTED.**

**DO NOT USE PRESERVATIVE FREE SALINE TO CLEAN AND DISINFECT YOUR LENSES.**

In the event of persistent eye irritation discontinue use, remove lens from eye and consult your contact lens practitioner. Do not swallow the solution. Keep all contact lens care products away

CooperVision®

from children. Do not remove cap from case during contact lens storage. Always replace bottle cap after use. Once bottle is open, discard any remaining solution after 3 months. Store between 4°C to 30°C. Do not rinse lenses or lens cases with water directly from the tap. Use before

expiration date printed on the carton and bottle.  Do not use if tamper-evident seal is broken.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and applicable competent authority.

INGREDIENTS: 0.005%w/v hydrogen peroxide, sodium chloride, sodium phosphates, water.

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